

# Efficacy of copper-impregnated wound dressing for wound healing in repeat cesarean section: A randomized controlled trial

Published March 2026



Asian Journal of  
Medical Sciences

Copper-impregnated dressings significantly improved post-cesarean wound healing scores compared with standard care, while also showing a lower trend in surgical site infections and shorter hospital stay. The study supports copper dressings as a safe, cost-effective option for improving healing outcomes in surgical wounds.

This randomized controlled trial evaluated 100 women undergoing repeat cesarean section, comparing copper-impregnated wound dressings with standard of care dressings.

The primary outcome was wound healing measured by the REEDA score — Redness, Edema, Ecchymosis, Discharge, and Approximation — on Days 7, 10, and 30 after surgery.

The copper dressing group showed significantly lower REEDA scores ( $p < 0.001$ ) at all assessed time points, indicating better wound healing.

- Additional outcomes: Lower surgical site infection (8% of the copper group compared with 20% in the control group)
- 0% readmission rate in the copper group (compared with 6% in controls)
- 2.5 days shorter mean hospital stay in the copper group (6.1 vs. 8.6 days)
- No adverse reactions were reported in the copper group.

“Copper-impregnated dressings improved wound healing, reduced SSI trend, and shortened hospital stay, suggesting a safe, cost-effective alternative warranting larger multi-centric evaluation.”

Agrawal, S.; Sharma, R.; Aggarwal, R.; Goyal, N. Efficacy of copper-impregnated wound dressing for wound healing in repeat cesarean section: A randomized controlled trial. *Asian Journal of Medical Sciences*, 17(3), 115–119, March 2026. <https://doi.org/10.71152/ajms.v17i3.5138>

Read the full paper online: <https://ajmsjournal.info/index.php/AJMS/article/view/5138>

# Efficacy of copper-impregnated wound dressing for wound healing in repeat cesarean section: A randomized controlled trial



Swarnima Agrawal<sup>1</sup>, Richa Sharma<sup>2</sup>, Richa Aggarwal<sup>3</sup>, Nisha Goyal<sup>4</sup>

<sup>1</sup> Postgraduate Resident, <sup>2</sup> Director Professor, <sup>3</sup> Professor, Department of Obstetrics and Gynaecology, <sup>4</sup> Associate Professor, Department of Microbiology, University College of Medical Sciences and Guru Tej Bahadur Hospital, Delhi, India

Submission: 04-01-2025

Revision: 04-02-2026

Publication: 01-03-2026

## ABSTRACT

**Background:** *Surgical site infection (SSI) is a frequent complication following cesarean section and contributes substantially to maternal morbidity. With increasing cesarean delivery rates and limited resources in public hospitals, there is a pressing need for cost-effective infection-prevention strategies. Copper, with intrinsic antimicrobial properties, has emerged as a promising adjunct in wound care. Aims and Objectives: The aim of this study was to evaluate the efficacy and safety of copper-impregnated wound dressings compared with standard gauze dressings in women undergoing repeat cesarean section, specifically assessing wound healing using the Redness, Edema, Ecchymosis, Discharge, and Approximation (REEDA) score, SSI incidence, hospital stay, and adverse effects. Materials and Methods: A randomized controlled trial was conducted at a tertiary-care government hospital, enrolling 100 women undergoing repeat cesarean section (50 copper dressing; 50 standard gauze). Samples of copper dressings were provided free of cost by MedCu Technologies. Baseline characteristics, comorbidities, and operative variables were comparable. Wound healing was assessed using the REEDA score on post-operative Days 7, 10 and 30. Additional outcomes included SSI incidence, readmission, resuturing, adverse reactions, and hospital stay. Data were analyzed using the Statistical Package for the Social Sciences version 20.0, with  $P < 0.05$  considered significant. Results: The copper group showed significantly lower REEDA scores on Days 7, 10 and 30 ( $P < 0.001$ ). SSI occurred in 8% versus 20% of controls ( $P = 0.08$ ), with fewer readmissions (0% vs. 6%). Mean hospital stay was shorter ( $6.12 \pm 4.97$  vs.  $8.64 \pm 6.93$  days,  $P = 0.006$ ). No adverse reactions were reported. Conclusion: Copper-impregnated dressings improved wound healing, reduced SSI trend, and shortened hospital stay, suggesting a safe, cost-effective alternative warranting larger multi-centric evaluation.*

**Key words:** *Copper-impregnated wound dressings; Surgical site infection; Repeat cesarean section; Wound healing; Redness, edema, ecchymosis, discharge, approximation score*

## INTRODUCTION

Surgical site infection (SSI) is one of the most frequent post-operative complications after cesarean delivery and remains a leading cause of maternal morbidity, particularly

in resource-limited settings. Given the high incidence of 23% to 38%<sup>1</sup> of SSIs associated with cesarean sections in India, along with the rising prevalence of these procedures, there is a critical need for cost-effective, accessible interventions to improve post-operative outcomes. Copper

### Access this article online

**Website:**

<https://ajmsjournal.info/index.php/AJMS/index>

DOI: 10.71152/ajms.v17i3.5138

E-ISSN: 2091-0576

P-ISSN: 2467-9100

Copyright (c) 2026 Asian Journal of Medical Sciences



This work is licensed under a Creative Commons Attribution-NonCommercial 4.0 International License.

### Address for Correspondence:

Dr. Swarnima Agrawal, Postgraduate Resident, Department of Obstetrics and Gynecology, University College of Medical Sciences and Guru Teg Bahadur Hospital, Delhi, India. **Mobile:** 91-9818041726. **E-mail:** swarnimaagrawal1998@gmail.com

**Table 1: Baseline characteristics of study participants (n=100)**

Age (years, mean±SD)	Copper-impregnated group (n=50)	Control group (n=50)	P-value
BMI (kg/m <sup>2</sup> , mean±SD)	28.08±2.19	28.94±3.28	0.381
Mean family income (Rs)	22.78±2.98	23.23±3.75	0.381
Booking status (Booked/unbooked)	24,780±7,115.06	22,880±6,723.82	>0.05
Gestational age (weeks, mean±SD)	76%/24%	54%/46%	0.021**
Elective repeat LSCS (%)	37.74±1.70	37.91±1.98	0.381
Previous LSCS (1/2)	18%	14%	0.585
Indication of cesarean (Maternal/fetal)	80%/20%	88%/12%	—
Comorbidities present (%)	68%/32%	56%/44%	0.216
	Anemia (22%), diabetes (2%), hypertension (6%), obesity (4%), and hypothyroidism (6%)	Anemia (18%), diabetes (6%), hypertension (10%), obesity (6%), and hypothyroidism (6%)	0.841
Mean duration of surgery (min)	70.60±16.34	72.40±11.35	0.158
Type of anesthesia (Spinal/General)	98%	92%	0.362
NICU admission (%)	18%	22%	0.803
Most common NICU cause (Respiratory distress)	12%	10%	—
Mean birth weight (kg)	2.68±0.59	2.68±0.60	0.973

SD: Standard deviation, LSCS: Lower-segment cesarean section, NICU: Neonatal intensive care unit, BMI: Body mass index

is a trace element with well-established antimicrobial and wound-healing properties. It acts by generating reactive oxygen species, disrupting bacterial membranes, and modulating angiogenesis and collagen synthesis. Laboratory and clinical studies have demonstrated the broad-spectrum antimicrobial activity of copper against Gram-positive and Gram-negative organisms, including multidrug-resistant strains.<sup>2</sup> This study evaluated the efficacy of copper-impregnated wound dressings specifically in patients undergoing repeat cesarean sections, a population at heightened risk for complications due to repeated surgical interventions.

### Aim

To study efficacy of copper impregnated wound dressings for wound healing in repeat caesarean delivery.

### Objectives

Primary objective: To compare wound healing by REEDA (Redness, Edema, Ecchymosis, Discharge, Approximation) score on post-op day 7, day 10 and day 30 in copper impregnated wound dressing group and conventional dressing group.

### Secondary objectives

1. To compare incidence of surgical site infections between copper impregnated wound dressing and conventional dressing group.
2. To study the side effects of copper impregnated wound dressing in caesarean wound healing such as erythema, itching, edema, etc.
3. To compare length of hospital stay, readmission rates and resuturing rates in both the groups.
4. To compare associated comorbidities in both the groups.

## MATERIALS AND METHODS

### Study design and setting

A randomized controlled trial was conducted at the tertiary care hospital of Delhi. Institutional Ethics Committee approval and informed consent were obtained.

### Participants

All pregnant women with period of gestation more than 28 weeks undergoing repeat cesarean delivery either elective or emergency were eligible. Exclusion criteria included patients with hepatitis B, C, human immunodeficiency virus or syphilis, any dermatological condition around surgical site, patients allergic to copper or Wilson's disease or any acute febrile illness.

### Intervention

Participants were randomly assigned to Group A (n=50, copper-impregnated dressing) and Group B (n=50, standard sterile gauze dressing). Dressings were applied immediately postoperatively, and wound healing was assessed on Day 7, 10, and 30.

### Outcomes and measurements

Primary outcome was wound healing assessed by Redness, Edema, Ecchymosis, Discharge, and Approximation (REEDA) score on Days 7, 10, and 30 postoperatively. Secondary outcomes included SSI incidence, readmission/resuturing rates, adverse events, and duration of hospital stay.


### Statistical analysis

Data were analyzed using the Statistical Package for the Social Sciences version 20.0. Continuous variables were compared using t-test or Mann-Whitney U-test and categorical variables using Chi-square or Fisher's exact test. P<0.05 was considered statistically significant.

## RESULTS

Baseline characteristics were comparable between groups (Table 1). Copper-impregnated dressings showed

significantly lower REEDA scores from Day 7 onward (Day 7:  $2.69 \pm 1.81$  vs.  $4.00 \pm 1.82$ ,  $P < 0.001$ ; Day 10:  $1.84 \pm 2.26$  vs.  $3.28 \pm 2.52$ ,  $P < 0.001$ ; and Day 30:  $0.57 \pm 0.96$  vs.  $1.24 \pm 1.00$ ,  $P < 0.001$ ) (Figure 1).

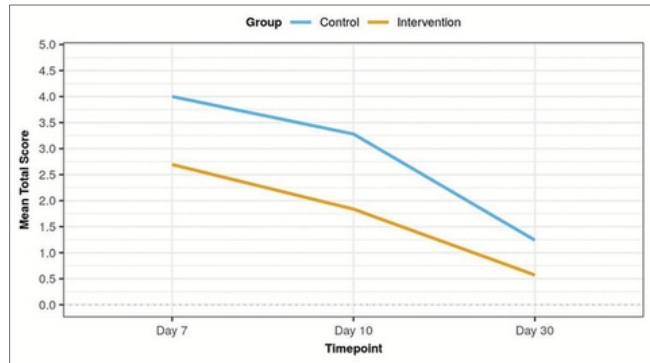

**Figure 1:** Comparison of redness, edema, ecchymosis, discharge, and approximation score over time

SSI occurred in 8% of the copper group versus 20% in controls ( $P = 0.084$ ) (Table 2), with fewer readmissions (0% vs. 6%) and significantly shorter hospital stay ( $6.12 \pm 4.97$  vs.  $8.64 \pm 6.93$  days,  $P = 0.006$ ). Ten superficial SSIs were observed (2 copper group, 8 control) and four deep SSIs (2 in each copper and control group). Common isolates included *Escherichia coli*, *Citrobacter freundii*, Methicillin-resistant *Staphylococcus aureus* (MRSA), Methicillin-susceptible *S. aureus*, *Acinetobacter baumannii*, and *Enterococcus*, (Table 3) with antibiotic sensitivity being imipenem and linezolid (30% each) (Table 4). All the cases were sensitive to imipenem. Three out of 4 (6%) in cases needed resuturing on Days 13, 16, and 25 while 7 out of 10 (14%) in controls

**Table 2: Comparison of SSI between both the groups**

Classification of SSI	Group			Fisher's exact test	
	Control (%)	Intervention (%)	Total (%)	$\chi^2$	P-value
Superficial	8 (80.0)	2 (50.0)	10 (12.5)	1.260	0.520
Deep	2 (20.0)	2 (50.0)	4 (28.6)		
Total	10 (100.0)	4 (100.0)	14 (100.0)		

SSI: Surgicals site infection

**Table 3: Comparison of organism isolated between both the groups**

Organism isolated	Group			Fisher's exact test	
	Control (%)	Intervention	Total	$\chi^2$	P-value
No organism	2 (20.0)	2 (50.0)	4 (28.6)	5.83	0.730
<i>Escherichia coli</i>	3 (30.0)	1 (25.0)	3 (21.4)		
MRSA	1 (10.0)	0 (0.0)	1 (7.1)		
<i>Acinetobacter baumannii</i>	0 (0.0)	0 (0.0)	1 (7.1)		
<i>Citrobacter freundii</i>	1 (10.0)	1 (25.0)	1 (7.1)		
<i>Enterococcus</i>	1 (10.0)	0 (0.0)	1 (7.1)		
MSSA	10 (100.0)	0 (0.0)	14 (100.0)		
Total		4 (100.0)			

MRSA: Methicillin-resistant *Staphylococcus aureus*, MSSA: Methicillin-susceptible *Staphylococcus aureus*

**Table 4: Comparison of antibiotic sensitivity between both the groups**

Antibiotic sensitivity	Group			Fisher's exact test	
	Control	Intervention	Total	$\chi^2$	P-value
Imipenem	7 (12.5)	2 (100.0)	3 (30.0)	5.833	0.800
Linezolid	3 (37.5)	0 (0.0)	3 (30.0)		
Cotrimoxazole	1 (12.5)	0 (0.0)	1 (10.0)		
Gentamicin	1 (12.5)	0 (0.0)	1 (10.0)		
Meropenem	1 (12.5)	0 (0.0)	1 (10.0)		
Tazopip	1 (12.5)	0 (0.0)	1 (10.0)		
Total	8 (100.0)	2 (100.0)	10 (100.0)		



**Figure 2:** Copper-impregnated wound dressing removal on Day 7

needed resuturing. No adverse reactions or rejections were reported.

## DISCUSSION

This randomized controlled trial evaluated the efficacy and safety of copper-impregnated wound dressings compared with standard gauze in women undergoing repeat cesarean section. The copper group demonstrated significantly lower REEDA scores from Day 7 onward (Figure 2) indicating faster wound healing, and a clinically relevant reduction in SSI incidence (8% vs. 20%), although this difference did not reach statistical significance. Mean hospital stay was also significantly shorter in the copper group, suggesting earlier recovery and potential resource savings.

The findings are consistent with previous reports by Arendsen et al.<sup>3</sup> and Melamed et al.,<sup>4</sup> who noted accelerated healing and lower infection rates with copper dressings, and by Wijetunge et al.,<sup>5</sup> who found comparable or superior results to silver dressings. These results highlight copper as a simple, low-cost, and locally applicable adjunct for post-operative wound care, particularly relevant in public hospitals and low-resource settings.

Limitations include the single-center design and modest sample size, which may limit external validity and the ability to detect smaller differences in secondary outcomes. Blinding was not feasible due to visible differences between dressings, introducing potential observer bias. Future multicentric studies with larger cohorts and cost-effectiveness analyses are warranted to strengthen the evidence base.

## CONCLUSION

These findings suggest that copper-impregnated dressings not only promote faster healing and earlier discharge but also may additionally help reduce colonization or infection with resistant organisms such as MRSA. The absence of

reported adverse effects further supports copper dressings as a safe, practical, and clinically relevant adjunct for post-operative wound care following repeat cesarean delivery.

## ACKNOWLEDGMENTS

The authors thank MedCu Technologies, Israel and BL Lifesciences, Okhla for providing copper-impregnated dressings free of cost for research purposes. We also acknowledge the contribution of nursing and operation theater staff at our Hospital for their assistance in patient care and follow-up.

### Strengths of the study

The study's strengths include its randomized controlled design ensuring baseline comparability, specific focus on repeat cesarean deliveries – a subgroup at higher surgical risk – and comprehensive outcome assessment covering wound healing, SSI, adverse events, and hospital stay. By addressing an under-researched population, this work contributes meaningful clinical data to existing literature, particularly from a resource-limited Indian context.

### Ethics approval

The study was approved by the Institutional Ethics Committee of the Hospital, Delhi, India. Written informed consent was obtained from all participants.

## REFERENCES

1. Arora A, Bharadwaj P, Chaturvedi H, Chowbey P, Gupta S, Leaper D, et al. A review of prevention of surgical site infections in Indian hospitals based on global guidelines for the prevention of surgical site infection, 2016. *J Patient Saf Infect Control.* 2018;6(1):1-12. [https://doi.org/10.4103/jpsic.jpsic\\_29\\_17](https://doi.org/10.4103/jpsic.jpsic_29_17)
2. Borkow G, Roth T and Kalinkovich A. Wide spectrum potent antimicrobial efficacy of wound dressings impregnated with cuprous oxide microparticles. *Microbiol Res.* 2022;13(3):366-376. <https://doi.org/10.3390/microbiolres13030029>
3. Arendsen LP, Thakar R, Bassett P and Sultan AH. The impact of copper impregnated wound dressings on surgical site infection following caesarean section: A double blind randomised controlled study. *Eur J Obstet Gynecol Reprod Biol.* 2020;251:83-88. <https://doi.org/10.1016/j.ejogrb.2020.05.016>
4. Melamed E, Kiambi P, Okoth D, Honigberg I, Tamir E and Borkow G. Healing of chronic wounds by copper oxide-impregnated wound dressings-case series. *Medicina (Kaunas).* 2021;57(3):296. <https://doi.org/10.3390/medicina57030296>
5. Wijetunge S, Hill R, Morris RK and Hodgetts Morton V. Advanced dressings for the prevention of surgical site infection in women post-caesarean section: A systematic review and meta-analysis. *Eur J Obstet Gynecol Reprod Biol.* 2021;267:226-233. <https://doi.org/10.1016/j.ejogrb.2021.11.014>

**Authors' Contributions:**

**SA-** Literature survey, prepared first draft of manuscript, implementation of study protocol, data collection, data analysis, manuscript preparation and submission of article; **RS-** Concept, design, clinical protocol, manuscript preparation, editing, and manuscript revision; **RA-** Review manuscript; **NG-** Review manuscript.

**Work attributed to:**

University College of Medical Sciences and Guru Teg Bahadur Hospital, Delhi-110095, India.

**Orcid ID:**

Dr. Swarnima Agrawal - <https://orcid.org/0009-0004-4042-5430>

Richa Sharma - <https://orcid.org/0000-0001-7132-684X>

Richa Aggarwal - <https://orcid.org/0000-0001-7672-0328>

Nisha Goyal - <https://orcid.org/0000-0002-8828-1139>

**Source of Funding:** The material (copper dressings) was provided by "MedCu Technologies Ltd. Herzliya, Israel" (Manufacturer) and "BL Lifesciences Pvt. Ltd. A-245, Okhla phase -1, Delhi" (Importer), free of cost, **Conflicts of Interest:** None.